


QuickStix™ Kits for TSWV and ToANV QuickGuide

Collect Sample

Fold the leaf (or stack two leaves) and punch once to collect two leaf sample punches.

If the cap does not punch through the leaves cleanly, tear excess leaf tissue away while holding the top closed.

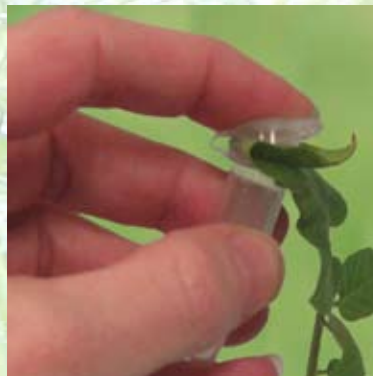

Test Procedure



1. Grind the tissue with pestle



2. Add 13 drops (~0.6 mL) of buffer and grind again



3. Add a test strip, wait 10 minutes

Test Interpretation

Read the result:



ANY pink Test Line is positive for the pathogen

A Control Line indicates the test has functioned properly